Vertrieb / Dystrybutor / Distributor:

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REV GA-22-01-CMS50D/2020-06

Contec Medical Systems Co., Ltd.
No. 112 Qinhuang West Street
Economic & Technical Development Zone
066004 Qinhuangdao, Hebei Province
P. R. China

Shanghai International
Holding Corp. GmbH
(Europe)
Eiffestr. 80
20537 Hamburg
Germany

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1	General	p. 14
2	Safety and responsibilities	p. 14
3	Modification of the product	p. 15
4	Before using the product	p. 15
5	Using the product	p. 16
5.1	Attaching the finger clip	p. 16
5.2	Switching the device on	p. 16
5.3	Declaration of conformity for other equipment	p. 17
	and systems – electromagnetic radiation	
5.4	Measuring principle	p. 17
6	Product maintenance/Reprocessing	p. 17
7	Accessories/Package content	p. 17
8	Replacement parts and repairs	p. 18
9	Disposal	p. 18
10	Explanation of symbols	p. 19

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1 General

- The device may only be used by persons familiar with the product.
- The Fingertip Pulse Oximeter can be used to measure a person's level of oxygen saturation and pulse rate via his/her finger.
- The product is suitable for use in hospitals, oxygen bars, socio-medical facilities, household environments as well as for medical supervision in sports.
- The pulse oximeter only plays a supporting role in patient diagnosis. It does not replace a medical diagnosis.
- These operating instructions reflect the state of the art at the time of publication.
- This product complies with the provisions of the Directive 93/42/EEC.
- The instructions for use comply with DIN EN 82079-1.
- The measuring accuracy has been confirmed in tests.
- The materials in direct contact with the patient's skin include medical silicone and ABS plastic and meet the normative requirements of ISO 10993-5 and ISO 10993-10.

2 Safety and Responsibilities

IMPORTANT! PLEASE READ CAREFULLY BEFORE USE
PLEASE KEEP FOR FURTHER REFERENCE

- Danger of explosion: The device must not be used in the proximity of flammable gas.
- The device must not be used together with MRI or X-ray CT systems.
- Check the sensor's measuring point constantly, otherwise measurements may be inaccurate.
- Check the product regularly for any visible damage. In case of damage please contact customer service.
- It may not be used as a monitoring device for any significant length of time since the oximeter has no warning function for blood oxygen saturation.
- The measuring point/measuring finger should be changed at least every 4 hours, by checking the condition of the patient's skin and the state of the patient's circulatory system.
- The blood oxygen saturation measurement process can be influenced by strong light. If necessary protect the sensor from light.

- Never hold the pulse oximeter under water, as fluid can enter into the device and the pulse oximeter will become damaged.
- Keep the pulse oximeter out of reach of children.
In case small parts such as battery compartment covers, batteries or retaining straps are swallowed, there is a risk of suffocation.
- The following may lead to incorrect results or malfunctions of the product:
 - Random movements by the patient
 - High frequencies produced by defibrillators
 - If the sensor's measuring point is located on the same artery or vein as a blood pressure cuff
 - If the patient has low blood pressure, very low contraction pressure, extreme anaemia or too low body temperature
 - Varnished nails
 - If the patient was treated with heart stimulants following cardiac arrest

3 Modification of the product

- Apart from the described maintenance instructions, the product may not be modified or repaired by the operator!
- The manufacturer's responsibility is limited to the dimensional and physiological suitability of the product.
- Improper use, modifications to the device or negligence will void any warranty. This also applies to unauthorised repairs. Normal signs of wear and tear on the device are also excluded.

4 Before using the product

- Store the device in a place with temperatures between -40°C and $+60^{\circ}\text{C}$ and a relative humidity of 5% to 95%.
- We recommend that you always store the device in a dry place.
Damp surroundings will limit the service life and may cause defects.
- Prior to each use, observe the reprocessing instructions as described in chapter 6.
- Attaching the carrying strap:
Guide the thin end of the carrying strap through the hole on the device.
Pull the thick end of the carrying strap through the loop that has formed and pull it closed with care.
- Inserting the batteries:
Insert 2 x AAA batteries observing the correct polarity.
Close the battery cover.
The device may become damaged if the polarity is incorrect.
Remove the batteries if the device is not used for any significant length of time.


5 Using the product

5.1 Attaching the finger clip

- Press the clip together to open.
- Insert the patient's finger with the fingernail facing upwards as far as possible between the rubber half-shells.
- Close the clip with care.

5.2 Switching the device on

- Press the on/off button on the front of the device once.
- The patient's finger should be held still during measurement and rapid movement should be avoided.
- Wait until the device finds a signal and then read the measured values from the display.
- You can select between 6 different display modes.
- If no signal is measured for more than 5 seconds, the device will switch off automatically.

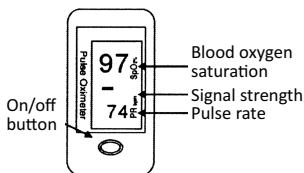5.3 Declaration of conformity for other equipment and systems – electromagnetic radiation

- For more information refer to www.droh.de.

5.4 Measuring principle

- The measuring principle of the oximeter is based on the application of the Lambert-Beer law, taking into account the absorption spectrum of reduced haemoglobin (HbR) and oxygenated haemoglobin in the visible and infrared ranges.
- During measurement different wavelengths of light (660 nm visible light and 940 nm near-infrared light) are passed through the patient's fingertip via a perspective sensor.
- The signal is measured by a light-sensitive sensor. The information acquired in this way is processed electronically and shown on the display.

6 Product maintenance / Reprocessing

- Do not sterilise or immerse in any liquid!
The device is not suitable for sterilisation.
- Clean the surface of the device with a disinfectant suitable for plastic before each examination.
- Please clean the rubber on the inside of the pulse oximeter as well as the patient's finger with medical-quality alcohol before use.
The rubber used is medically safe, contains no toxic substances and is skin-friendly.
- Make sure that no moisture enters into the device through any openings.
Let the pulse oximeter dry completely before using again.

7 Accessories / Package content

- Fingertip Pulse Oximeter
- Two batteries (AAA)
- Operating Instructions
- Carrying strap

8 Replacement parts and repairs

- Please contact customer service in case any problems arise other than the following.

Problem	Cause	Solution
SPO ₂ or pulse rate not displayed.	1. Finger has not been correctly inserted. 2. The patient's blood saturation is below the measurement range. 3. Possibly nail varnish is applied.	1. Insert finger again, this time correctly. 2. Try a number of times. Otherwise consult a doctor or hospital for an exact diagnosis. 3. Remove any nail varnish.
SPO ₂ or pulse rate displayed at irregular intervals.	1. Finger has not been inserted far enough. 2. The patient's finger is trembling or the patient is moving too much.	1. Insert finger again. 2. Avoid any movement.
The oximeter cannot be switched on.	1. The batteries are low. 2. The batteries are incorrectly inserted. 3. The oximeter is perhaps damaged.	1. Replace batteries. 2. Insert batteries again. 3. Contact customer service.
The display switches off suddenly.	1. Device switches off automatically if there is no signal for 5 seconds. 2. Battery voltage is too low.	1. Switch device on again. 2. Replace batteries.

9 Disposal

- Safe disposal is carried out in accordance with current local regulations.
- Please dispose of the batteries separately. Please remove the batteries from the device as described in Chapter 4. Batteries must not be disposed of with general waste. You can return used batteries to **Wolfram Droh GmbH** free of charge. As a consumer you are legally obliged to return used batteries.
- Batteries containing hazardous substances are marked with a symbol consisting of a crossed-out waste bin and a chemical symbol (Cd, Hg or Pb), indicating the heavy metal which is the reason for classification as containing hazardous substances:

10 Symbolerklärung / Objaśnienie symboli / Explanation of symbols

		
 Deutschland Kennzeichen	Symbol Niemiec	Symbol for Germany
 Polen Kennzeichen	Symbol Polski	Symbol for Poland
 Groß-Britannien Kennzeichen	Symbol Wielkiej Brytanii	Symbol for Great Britain
 Gebrauchsanweisung beachten	Przestrzegać instrukcji użytkowania	Observe the operating instructions
 Warnhinweis	Wskazówka ostrzegawcza	Warning
 Hersteller	Producent	Manufacturer
 EU-Repräsentant	Przedstawiciel UE	EU Representative
 CE-Symbol	Symbol CE	CE symbol
 Batterie-Entsorgung	Utylizacja baterii	Battery disposal
 Typ BF_nach EN 60601-1	Typ BF według normy EN 60601-1	Type BF according to EN 60601-1