SORIMEX

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EN Ultrasound medical gel

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SORIMEX sp. z o.o. (dawniej SORIMEX sp. z o.o. sp. k.)
Równinna 25, 87-100 Toruń, Poland
office@sorimex.eu ♦ www.sorimex.eu

KAF GRUP SAĞLIK HİZMETLERİ İNŞAAT SANAYİ VE TİC.LTD.ŞTİ.
Atakent Mh. 221. Sk. No:3A Rota
Office A Blok Kat: 14 D:82-83 K.ÇEKMECE/İSTANBUL,
Türkiye, Made in Turkey

Rev. 2025-05-19

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PL

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USG - SR09, SR10, SR11, SR12.

Instruction for use

Instruction for medical gel for ultrasound:

1. Description

High-quality ultrasound gel that ensures excellent transmission of ultrasound waves and precise imaging. Its optimal viscosity eliminates air bubbles and prevents drying during examinations, ensuring a stable signal. It washes off easily, does not damage probes, and is safe for the skin, providing full user comfort. A small amount of blue dye is used to aid in precise application and control of skin coverage without causing irritation.

2. Directions for use

Apply an appropriate amount of gel to the skin. After the examination, remove the gel using a cloth or towel.

3. Contraindications

The gel is intended for external use only and should not be used in the following cases:

- In any invasive procedures where the device passes through tissue, e.g., needle aspiration, needle localization, tissue biopsy, or other surgical procedures.
- In any procedure performed on damaged skin or near it, e.g., a fresh postoperative site or open wound.
- In patients with immune deficiency or those undergoing immunosuppressive therapy.
- In newborns or severely ill pediatric patients.
- On mucous membranes.
- In critically ill patients or those under special precautions regarding contact, airborne, or droplet transmission.

4. Storage conditions

Store at temperatures from 0°C to 35°C, away from direct sunlight, in a cool and dry place.

5. Ingredients

Deionized water, carbomer, triethanolamine, monopropylene glycol, methylchloroisothiazolinone, Brilliant Blue FCF (odorless and water-soluble dye, safe for the skin, used in food, pharmaceuticals, and cosmetics).

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